
Quick Reference Guide

Masimo SafetyNet™ App

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

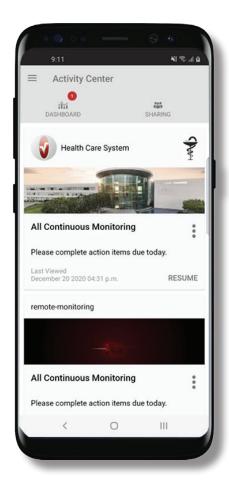

> Download and install the Masimo SafetyNet App from:

Google Play



The App Store



> Launch the app and follow the step-by-step instructions to create an account, engage in a shared CareProgram™, and begin collecting and sharing data.

For further assistance or questions visit: www.masimo.com/masimo-safetynet

©2021 Masimo Corporation. All rights reserved. All other trademarks and registered trademarks are property of their respective owners.



300212/LAB-10206C-0221

Log In, Sign Up, Location, Notifications


>Sign Up

- Sign up for an account using and email address or phone number.
- If you received a CareProgram from a health provider, you must sign in with the email or phone number used to send your invitation.



> Ensure Location and Notifications are allowed for proper app function.

Location

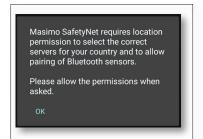


ALLOW

Notifications



www.masimo.com www.masimo.com 2

Health Permissions (iOS Only)

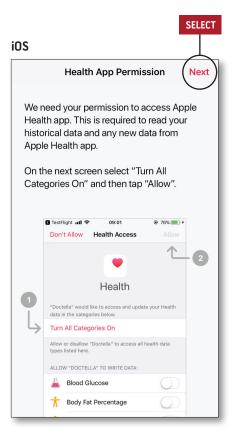

> Follow the on-screen instructions:

To allow Masimo SafetyNet to access
and gather health data from the
device.

Activity Center

> Menu

View the App Menu.

> Dashboard


- View the Dashboard screen, which shows the user's health data.
- View health data shared by other users.

> Sharing

- Displays the healthcare providers, caregivers, and family members with whom the user is currently sharing data.
- Share data with another healthcare provider, caregiver, or family member.

> CareProgram


 Allows user to view or delete a CareProgram.

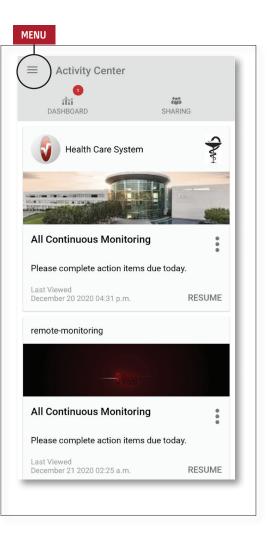

3 www.masimo.com www.masimo.com 4

Menu

• The Menu can also be viewed by swiping across the screen from left to right.

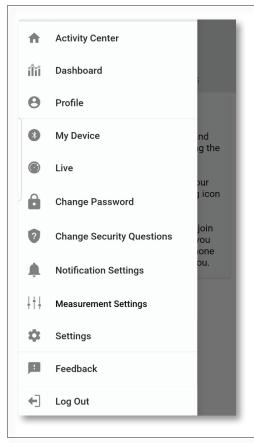

> Feedback

 Allows the user to send an email to Masimo SafetyNet support.

> Log Out

5

 Allows the user to log pout of the App.

Activity Center

• Displays the Activity Center screen.

Dashboard

Displays the Dashboard screen.

Profile

- Displays the user profile information.
- Allows the user to modify the profile information.

> My Device

 Allows the user to add a device for monitoring.

Live

 Allows the user to view data from the connected monitoring device.

Change Password/Change Security Questions

 Allows the user to change the password and/or security questions in case of lost password.

Notification Settings

 Allows the user to change the app notification and reminder settings.

Measurement Settings

 Allows the user to change the units of measurement.

> Settings

- Displays the user ID.
- Allows user to enable or disable
 Touch ID or Face ID to log into app.
- Displays app information.

Dashboard

> Invite


 Allows the user to invite others to share information through the app.

> Share


 Displays the healthcare providers, caregivers, and family members with whom the user is currently sharing data.

> Edit

- Allows the user to rearrange data on the Dashboard.
- > View data for the last 30 days or a weekly summary.
- > Select a parameter card to view trends, date ranges, and additional data.

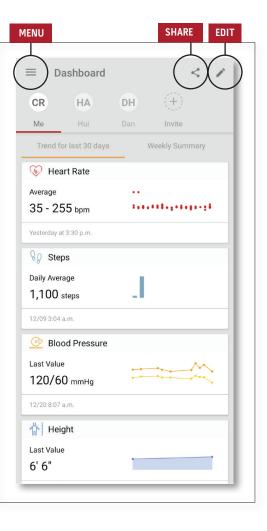

CareProgram

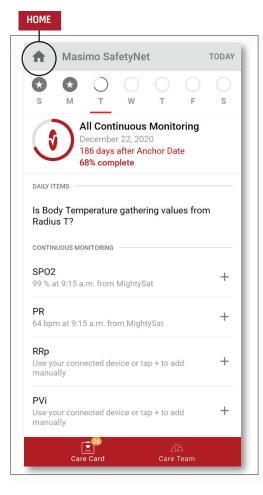

7

> Care Card

- View the Care Card data.
- > Home
- Return to the Activity Center.
- > Learn
- Located at the bottom of the CareProgram, view additional information provided by a caregiver.
- > Care Team
- View the list of shared healthcare providers, caregivers, and family members.

Sharing

> Displays the list of shared healthcare providers, caregivers, and family members.

>Add

- Share with additional healthcare providers.
- Share with additional caregivers or family members.
- > Swipe a shared individual to remove or edit privileges.



ADD

Who do you want to share your health data with?

Healthcare Provider

Caregiver or family member

CANCEL