Masimo SafetyNet™

Radius PPG™


Warning: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

GET TO KNOW YOUR SYSTEM



Masimo SafetyNet App

Masimo SafetyNet displays Oxygen Saturation, Pulse Rate, Respiration Rate and other data from Radius PPG. It also sends data to your care provider.

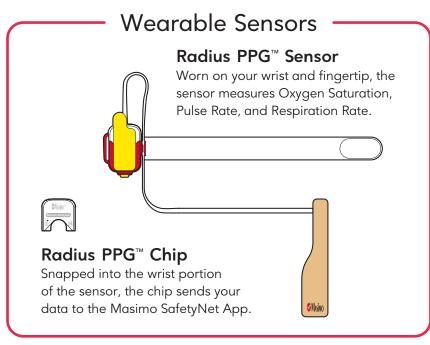


Getting Started

1 Set-up Masimo SafetyNet App

Follow instructions from the email or text message invitation to download the Masimo SafetyNet App and create an account.

2 Apply Sensor

See instructions on the back showing how to apply the sensor to your wrist and finger and insert the chip.

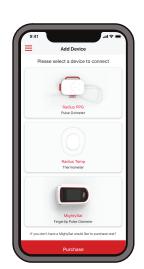

? Pair Sensor

Under Masimo SafetyNet App **Settings**, enter the **Add Device** menu, and follow instructions for pairing the Radius PPG sensor to your phone.


For additional resources, including videos and instructions, visit **mymasimo.com**






How to Apply your Sensor

.1 Sensor Set Up

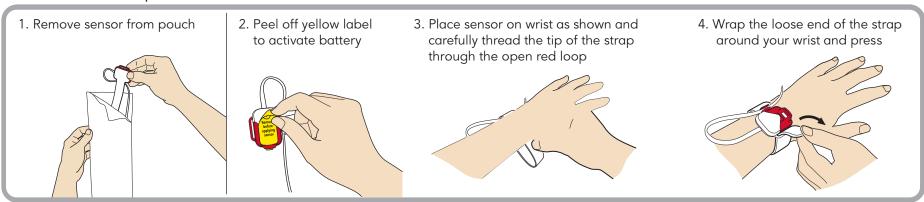

2.2 Apply Finger Senso

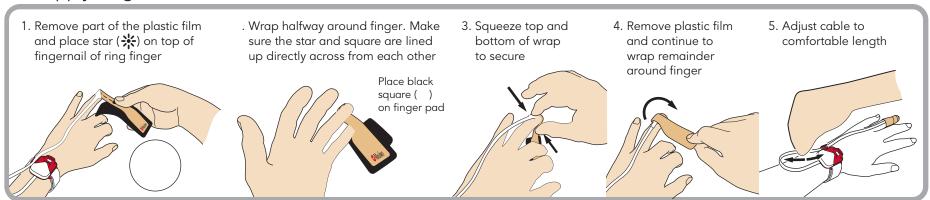

2.3 Insert Chip

